INSTRUCCIONES DE USO CALCINABLES

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IFU-CAL v01 2015 PT

INSTRUCTIONS FOR THE USE OF CASTABLES

1. Product description

Castables are elements made of POM-C (Polyacetal (copolymer)). This material is used because of its perfect melting characteristics, extraordinary evaporation capabilities and low hygroscopic coefficient.

Castables are machined with high precision because they greatly influence the quality of the final prosthesis.

All castables usually have an open orifice that enables fixing the resulting prosthesis to the implant or abutment.

2. Planned use:

Castables are the support for creating the final prosthesis.

3. Directions for use

Once the castable is screwed onto the laboratory replicate, dental materials are used to sculpt the shape and form of the prosthesis.

Once this process is completed, the "lost wax" system is used to create the final prosthesis.

Phases:

- 1.- add sufficient wax to create a layer that will compensate for expansion of the castable
- 2.- press gently to prevent any deformation
- 3.- pour parts of a size and shape that favours filling any cavities and prevents the formation of air bubbles
- 4.- use high fluidity alloys for complex models.

4. Cleaning, disinfection and sterilisation

In the event of any packaging defect, the product should not be used and the manufacturer notified immediately.

All BIONER castables are supplied clean and appropriately packaged. As they are not inserted in the mouth and their use is restricted to the laboratory, they require no sterilisation.

5. Warning

It is important to bear their melting point in mind and program the furnace accordingly to achieve best results.

6. Precautions

It is especially important to ensure that each reference is used in combination with dimensionally compatible elements as indicated on the product label.

7. Compatibility

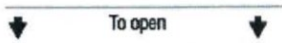
BIONER castable family references must be used in conjunction with compatible components. Carefully examine the components available in the company product catalogue for suitability before insertion.

8. Validity

These instructions for use correspond to the current version of the castable family references indicated at the foot of the document.

9. Storage

BIONER products do not require any special storage conditions even though it is recommended not to expose them to conditions that could affect the product, its packaging or label, such as extreme temperature and humidity, knocks, etc.

ISO 15223 Symbol	Description
	Indicates the direction for opening the blister
	BIONER SA logo type
REF	Catalogue reference
LOT AOFAAXXXXX	Batch code
 DD/MM/YY	Date of manufacture
	Refers to the instructions for use
	Do not use if package is damaged
Family	Name of the product family
Mat = POM (Polyacetal (copolymer))	General indication of the material used
q=1	Indicates the number contained in each blister pack
	Non-sterile
	QR code (reference and batch)
	Do not reuse

	Consult warnings and precautions in instructions for use
	Manufacturer
	EC Marking

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Name BIONER, S.A.